RIDDER

Assistent

MD

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CE

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User manual for walker A134110-453


Dear customer,
thank you for choosing our product. Before use, please read the safety instructions carefully and keep the manual for further use. The manufacturer assumes no liability for incorrect assembly or improper use of the walker.

Delivery content


1 x pre-mounted walker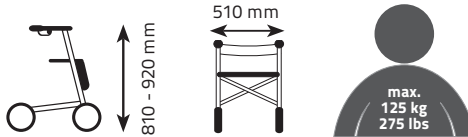

1 x fabric shopping bag
(max. capacity: 9l, max. load limit: 5 kg / 11 lbs)

1 x back belt

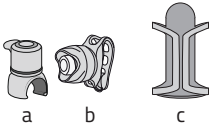1 x bell (a)
1 x LED light (b)
1 x cup holder (c)

1 x mounting instructions and user manual

Technical data:

Item no: A134110-453

Model: Rollator Comfort Plus, black, incl. bell, walking stick holder, cup holder and LED light

Max. dimension of the mounted walker (WxHxD): approx. 510 x 930 x 750 mm

Min. dimension of the folded walker (WxHxD, without accessories): approx. 210 x 850 x 500 mm

Weight of the walker: approx. 8 kg (17.6 lbs)

Max. user weight: 125 kg (275 lbs)

Max. load limit of the bag: 5 kg (11 lbs)

Seat height: approx. 570 mm

Dimensions of the seat: approx. 450 x 250 mm

Grip height: approx. 810 - 920 mm

i NOTE!

Please open the packaging carefully, so that the product does not get damaged. Check the full delivery content for completeness and intactness. If you find any defects, please contact the service address given.

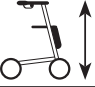
1. Bell
2. Hand grip
3. Back belt
4. Hand brake
5. Walking stick holder
6. Frame (brake cables placed at the inner side)
7. LED light
8. Seat
9. Drawstring for folding
10. Fabric shopping bag with belt
11. Swivel wheel with reflectors
12. Cup holder
13. Button for height adjustment
14. Tipping aid
15. Rear wheel with brake

i NOTE!

Stability notes:

The walker supports an individual weight up to 125 kg (275 lbs), INCLUDING the loading of the basket. The bag has a weight limit of 5 kg (11lbs).

 WARNING!	Warning, danger! Failure of respect the danger can result in serious injury or death.
 CAUTION!	Caution, danger! Failure of respect the danger can result in minor or moderate injury.
 NOTE!	Mind possible property damages
	Min. and max. height of the walker
	Max. width of the walker
	Max. user weight
	Can be used indoors and outdoors
	Follow the instructions!
	Distributor
	Date of manufacture
	Declaration of conformity (CE conformity)
	Medical product
	Mind hand injuries
	Disposal of the packaging
	Disposal of the old device (LED light)
	Disposal of batteries
2 years	Implied warranties

Safety instructions

WARNING!

- The product is not designed for the use by persons with limited physical, sensory or mental abilities. It must not be used in case of a lack of knowledge or experience, unless instructions have taken place in which the risks were explained and the persons concerned are supervised by a caregiver.
- Any serious incidents related to the product must be reported to the manufacturer and the relevant authority.
- This walker is not a toy! Do not leave children use this product unattended.
- Do not use the walker under the influence of alcohol, medication or other drugs that affect your physical, mental and motor skills.
- All wheels must have constantly contact with the ground. This ensures the correct alignment of the walker. Therefore, do not use the walker on uneven grounds like sand or gravel. In particular, pay attention to uneven places such as gully tops. If possible, avoid driving over them as the wheels could get stuck.
- Do not use the walker on iced and snow covered or on very wet grounds. Slippery surfaces affect the slip resistance of the wheels in a negative way.
- Do not use the walker in bad visibility conditions (e.g. in fog or at night).
- If you use the walker outside, please use, if possible, the sidewalk. Wear striking and reflecting clothes. If necessary, please turn on the LED light.
- Do not use the walkers on stairs or escalators.
- Transport things solely in the fabric bag provided. Pay attention to the max. load weight and max. capacity.
- Do not use the walker as a vehicle while you are sitting on it. Before using the seat, the parking brake must be activated.
- When adjusting the height of the handles, make sure that you take up an upright and comfortable body posture. In order to prevent bad posture, let yourself be instructed by a professional for medical and rehabilitation products.
- Make sure that the handles on both sides are set on the same height and that the screws are locked correctly.
- Before every use, check the function of the brakes and make sure that the side tubes and the seat are locked properly.
- Check regularly if the walker has damages or wear.
- Have defective components replaced immediately and do not use the walker until it has been completely repaired. Do not make any changes to the product.
- Defective components must be replaced immediately and the walker must not be used until it has been completely repaired. Do not make any changes to the product.
- **Do not use the product if it is damaged!**
- Always respect the maximum load limits of the walker and its accessories. Never exceed this limit, otherwise the function and stability of the product can not be guaranteed.
- Do not stand on the walker and do not use it as a stand-up or ascending aid.

- Since the walker is equipped with a folding function, there is always, and especially in the process of unfolding and folding, the risk of being bruised or trapped.
- Always take care to protect your fingers and your hands from being trapped.
- Risk of burns! Do not expose the walker to direct sunlight or other extreme heat sources as the material can heat up quickly.
- Keep packaging material away from children.

Safety instructions for the LED light with battery

WARNING!

LED light

- The light source of this LED light is not exchangeable. If it is defective, the entire lamp must be replaced and the defective lamp properly disposed of.
- Never change the lamp yourself!
- Only use the lamp if it is properly attached to the rollator.
- The LED light may only be used on the rollator provided for it and may not be used for purposes other than intended.
- Do not drop the lamp or subject it to mechanical pressure.
- Never use a magnifying glass or glasses with strong optical magnification within the light cone of the LED. **Fire hazard!**
- Do not look directly into the light of the lamp.

WARNING!

Batteries

- Remove the batteries from the lamp as soon as it is used up or if you will not be using it for a long time.
- Warning: Always keep batteries out of the reach of children.
- Do not charge, reactivate or short-circuit batteries.
- Never open a battery!
- Do not use damaged batteries!
- Do not heat the batteries! Risk of explosion!
- Be careful with leaking batteries: Dispose of them immediately according to regulations. Protect your skin from leaked acid.
- If skin comes into contact with the acid, wash the affected area thoroughly with soap and water. If the battery acid gets into your eyes, rinse it out thoroughly with water and consult a doctor immediately.
- Keep batteries out of the reach of children. Swallowing batteries can have far-reaching health effects and even lead to death. If a battery is swallowed or otherwise got into the body, seek medical advice immediately.
- Always observe the polarity when inserting batteries.
- Always use batteries of the same type. Never mix new and old batteries.

Purpose and function

The structural design of the Rollator Comfort Plus supports the user when moving in terms of stability and balance. This improves individual mobility, compensates for unsteady gait and prevents falls. It serves as a walking aid for people with restricted mobility with a maximum body weight of 125 kg.

The rollator is only intended for private use and is not intended for commercial use.

The rollator is designed for indoor and outdoor use.

In addition, it offers a seat to counteract exhaustion caused by the effort of walking.

The rollator is designed for use on firm and level ground. Use on ice, snow and wet surfaces is prohibited.

indication

The use of the Rollator Comfort Plus is suitable for:

- Difficulty walking due to limitation of active or passive movement apparatuses
- Coordination difficulties in the running phase
- reduced resilience

contraindication

The use of the Rollator Comfort Plus is not suitable for:

- Perceptual disorders
- severe balance disorders
- a user weight of more than 125 kg

Intended Use

Observe the safety and warning notices in the installation and operating instructions directions. The rollator supports the user when transporting objects in the fabric bag provided for this purpose. The fabric bag has a capacity of approx. 9 liters and can be loaded with a maximum of 5 kg. Only use the rollator as described in the user manual. The LED light is used for better visibility. It does not represent lighting within the meaning of the road traffic regulations.

The recommended service life of the rollator is a maximum of 5 years. (this information assumes compliance with the intended handling, care and safety instructions).

Storage:

For storage, the parking brake should always be activated.

Store the walker in a dry place by room temperature (approx. 20 ° C). Protect it from frost, direct sunlight or other weather influences. Keep the product out of the reach of children.

Cleaning and care:

Please note the cleaning and care instructions. Improper operation can damage the product.

Clean the walker gently with water and a standard commercial cleaner. Then dry the product with a cloth.

Do not use any aggressive cleaning agents and always make sure not to expose the walker to a hard water jet (e.g. from a high pressure cleaner), as this could damage small parts or the surface of the product.

Maintenance:

Before each use, check the functionality of the walker. The following parts require your special attention:

- Brakes
- Wheels
- Screw connections
- Overall appearance of the product
- Locks

Disposal of the walker:

Please dispose the product in accordance with the applicable regulations via an approved waste disposal company or via the municipal waste facility. If necessary, inform yourself about correct and environmentally friendly disposal.

**Disposal of the packaging:**

Please dispose the packaging environmentally friendly. Make sure that you separate the waste properly.

**Disposal of electrical components (LED-lights):**

Every consumer is legally obliged to have electrical devices disposed of separately from household waste via an approved disposal company for electrical devices and thus to avoid environmental damage. Please remove the batteries before disposing of the item and dispose of them separately.

**Disposal of batteries:**

Batteries must not in the household trash! Consumers are legally obliged to hand them in an appropriate collection point or at the dealer so that environmentally friendly disposal can take place.

Commissioning


CAUTION!


Risk of injury from getting trapped during the entire assembly process!

The walker gets delivered in the folded state.


Unfold the walker: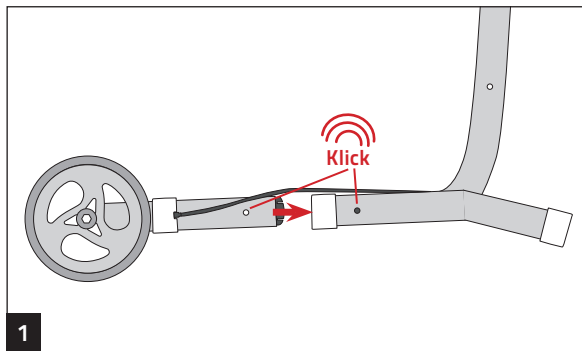

1 Move the strut of the rear wheels into the correct position and then press the two buttons on the sides of the strut so that the strut engages in the hole the hole provided in the frame.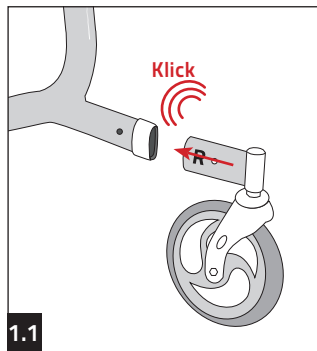

1.1 Repeat this procedure with the front wheels while observing the markings (R) and (L) on the struts.

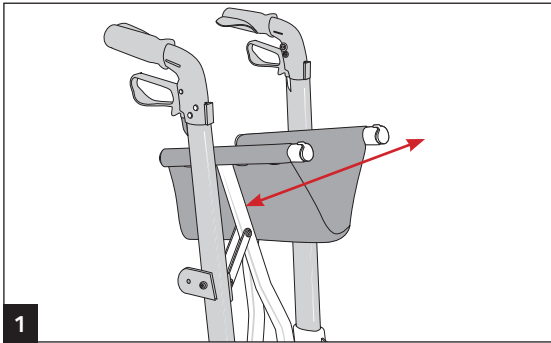


1 unfold the rollator as wide as possible.

2 The holders under the seat must snap into the small struts on the frame.



3 Now push the back strap from above onto the device on the handles until they are at the stop.

4 Eyelets (A) are attached to the right and left of the bag. Put the eyelets of the bag (A) on the left and right over the side tubes of the seat and then hook the loop into the barbs (B) attached to the tube to prevent the bag from slipping out.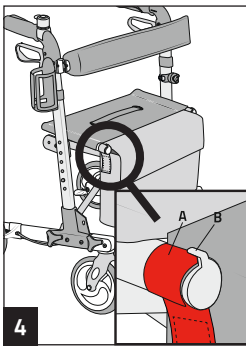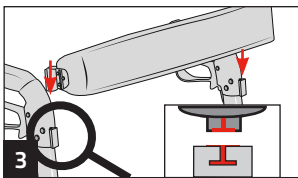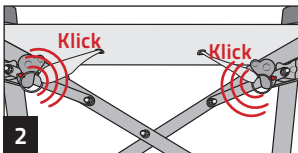

Mount bell:

5 Attach the bell to one of the handles depending on whether you are right or left handed. There are depressions (A) on the left and right of both handles, into which the bell frame must engage. These ensure a secure hold of the bell on the handle. Mount the bell so that you can press the bell with your thumb.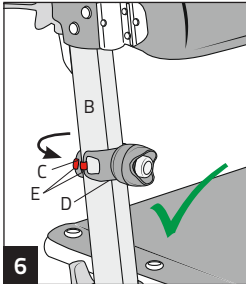
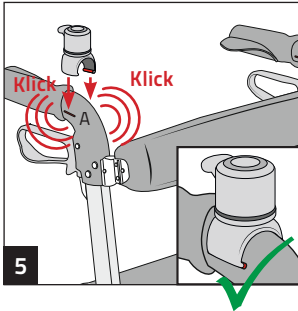


Mount LED light:

WARNING!

Only use the light on the rollator. The LED light is used to make the rollator more visible to other road users, but only to a limited extent to brighten up the area.

7 Attach the LED light to any aluminum tube of the rollator handles (B). Make sure that the light can be seen by oncoming traffic or pedestrians. Pull the long side of the rubber band (C) far enough so that it can be stretched around the aluminum tube (B) and hooked into the hook (E) provided on the short side of the rubber band (D).

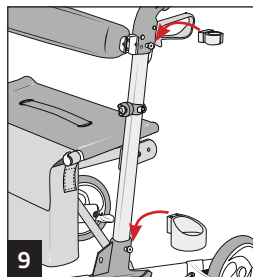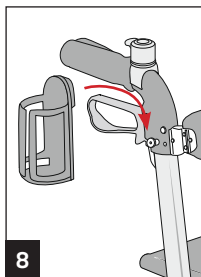Mount cup holder:

8 Insert the cup holder into the jig on the right handle and push down as far as it will go. Then you can use it.

Mount walking stick holder:

9 Fix the drip tray in the hook provided for this purpose on the lower part of the left strut and the clip for locking the stick shaft in place at the top of the left handle.

Adjust handle height:

i NOTE!

The handle height of the rollator can be adjusted in 6 steps using the locking knobs. Assume an upright but comfortable position and let your arms hang loosely. In this position, your hands should be able to grip the handles of the rollator **1**.



! WARNING!

Incorrectly adjusting the handle height can lead to poor posture and other injuries **2 + 3**. Only use the rollator with the handle height set correctly for you.

Make sure that both handles are locked at the same height. Depress the knob to loosen the grip and lock at a height that is comfortable for you. The handle must engage at the same height on both sides. Note that the handle must always be properly locked in one of the 6 heights provided **4**.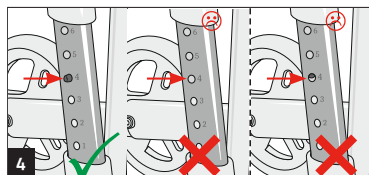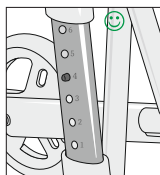

Proper use in everyday life:

! WARNING!

For best support, keep the rollator close to your body. Only then is safe walking guaranteed. If you move the rollator too far away from your body, it can easily roll away and you may not be adequately supported.

Use of the LED light:

Press the light's transparent bezel to operate the LED light. Choose from different modes by pressing the icon:

- 1. constantly glowing
- 2. blinking fast

- 3. blinking slowly
- 4. off

Apply the brakes:


WARNING!

Always apply the right and left brakes at the same time so that they do not wear out unevenly. Check the brakes for correct setting and functionality before each use. The parking brake must always be activated before using the seat. Stop using the walker if the brakes can no longer be adjusted correctly.

The brakes must be readjusted from time to time due to wear.


If the braking performance decreases or is uneven, you have to re-tighten the brakes.

No further training is required to use the rollator.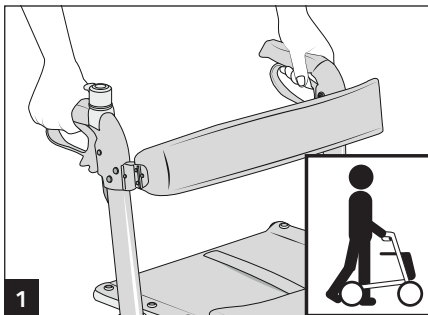

1 Hand-brake (while walking)

Pull both brake levers up towards the handlebar at the same time and with the same intensity. To loosen it, just let it loose again. The harder you pull, the greater the braking power. If the braking power is too low despite firm pulling, the brake may have to be re-adjusted.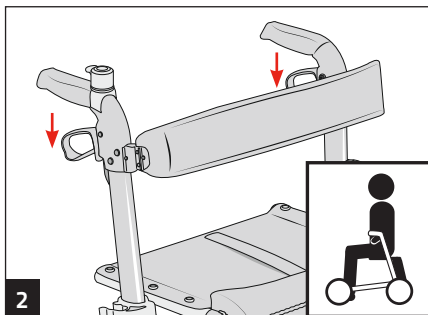

2 Parking-brake (while sitting)

Press both brake levers down until they lock and no longer snap back. Only then are the wheels fixed and the walker cannot roll away. To release, simply pull it back into the middle position with light pressure.

Use of the seat:

CAUTION!


Only use the seat when the parking brake is activated. Do not being pushed while using the seat or push anyone else. There is a risk of falling! The seat must only be used by one person at a time. Do not use the walker if the ground is smooth or slippery.

Changing the batteries from the LED-light:


2x lithium button cell CR 2032, 3V each

⚠ WARNING!


Please note the safety instructions for the LED-light and batteries!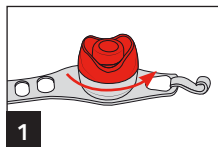


Remove the LED light from the walker. Rotate the top of the LED housing right round so it loosens and remove it.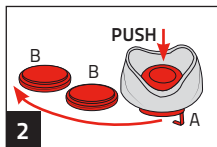

Now push the light (A) out of the housing and remove the two batteries (B). Pay attention to a professional disposal.

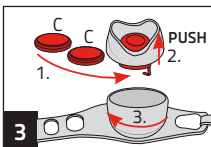
1. Insert the new batteries (C). Pay attention to the polarity.
2. Then insert the lamp back into the housing and
3. close it.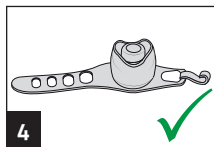


Now you can attach the light to the walker again as described.


Folding the rollator: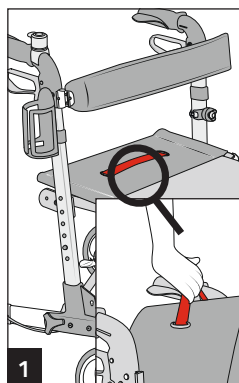

1

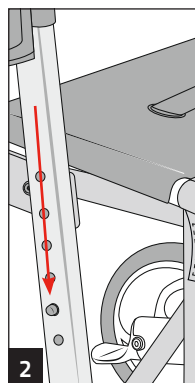2

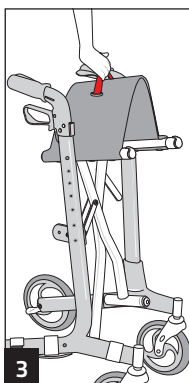
3

1 Pull the seat up on the grip tab so that the seat disengages from the bracket and the rollator contracts. Then use the clamp to lock it to the seat tubes.

2 If necessary, set the handle height to the lowest level to save even more space.

3 Transport the folded rollator using the handle bar.

Austausch Bremsblock:

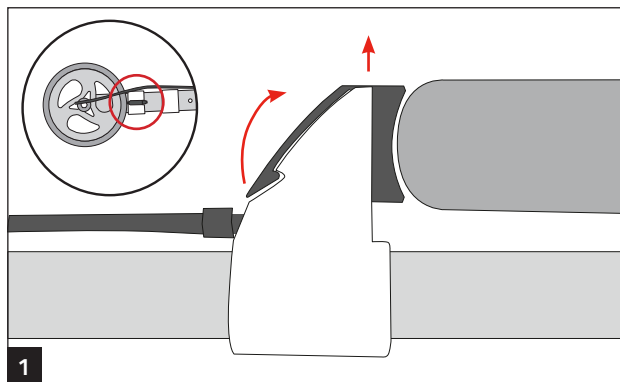1

1 Pull the tab of the brake (16) upwards to release it from the anchorage. Then pull the tab backwards to remove the entire brake block.

2 Insert the new brake block completely into the bracket until it clicks.

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RIDDER GmbH